


Head Routine / First Time Seizure

Application Examples: first time seizure, headache

Sequence	FoV	Thickness	Gap
SAGITTAL T1	24cm	4 mm	1 mm
AX T2 TSE PROPELLER	24cm	4mm	1 mm
AX T2 FLAIR PROPELLER	24cm	4mm	1 mm
AX SWAN	24cm	4mm	1 mm
AX T1	24cm	4mm	1 mm
AX DWI (B-0, B-1000)	24cm	3mm	1mm
INJECT CONTRAST (1mmol/Kg)			
SAG CUBE FLAIR WITH GAD (<i>RECON AX & COR 1 x1</i>)	24cm	1.2mm	NO GAP
SAG T1 FS SPGR WITH GAD	24cm	4mm	1mm
AX T1 FS SPGR WITH GAD	24cm	4mm	1mm

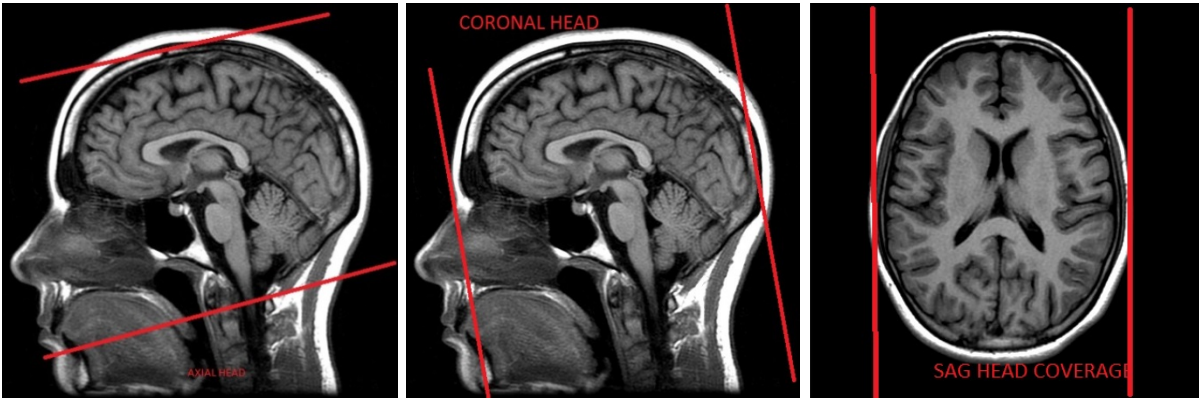

AXIAL Coverage

COR Coverage

SAG Coverage